


C:\Backup 16-03-2020\SW-Tek\Marina 2300\2300 82x204\

Signed:	ES	Date:	02-2011
Revised:	8 / KA	Date:	24-03-2020
Revised:	- /	Date:	-

## Overview parts Marina Top frame

Copyright reserved by law	Unit of measurement: mm	Lopital Nederland bv	Type nr: 61002350
---------------------------	-------------------------	----------------------	-------------------

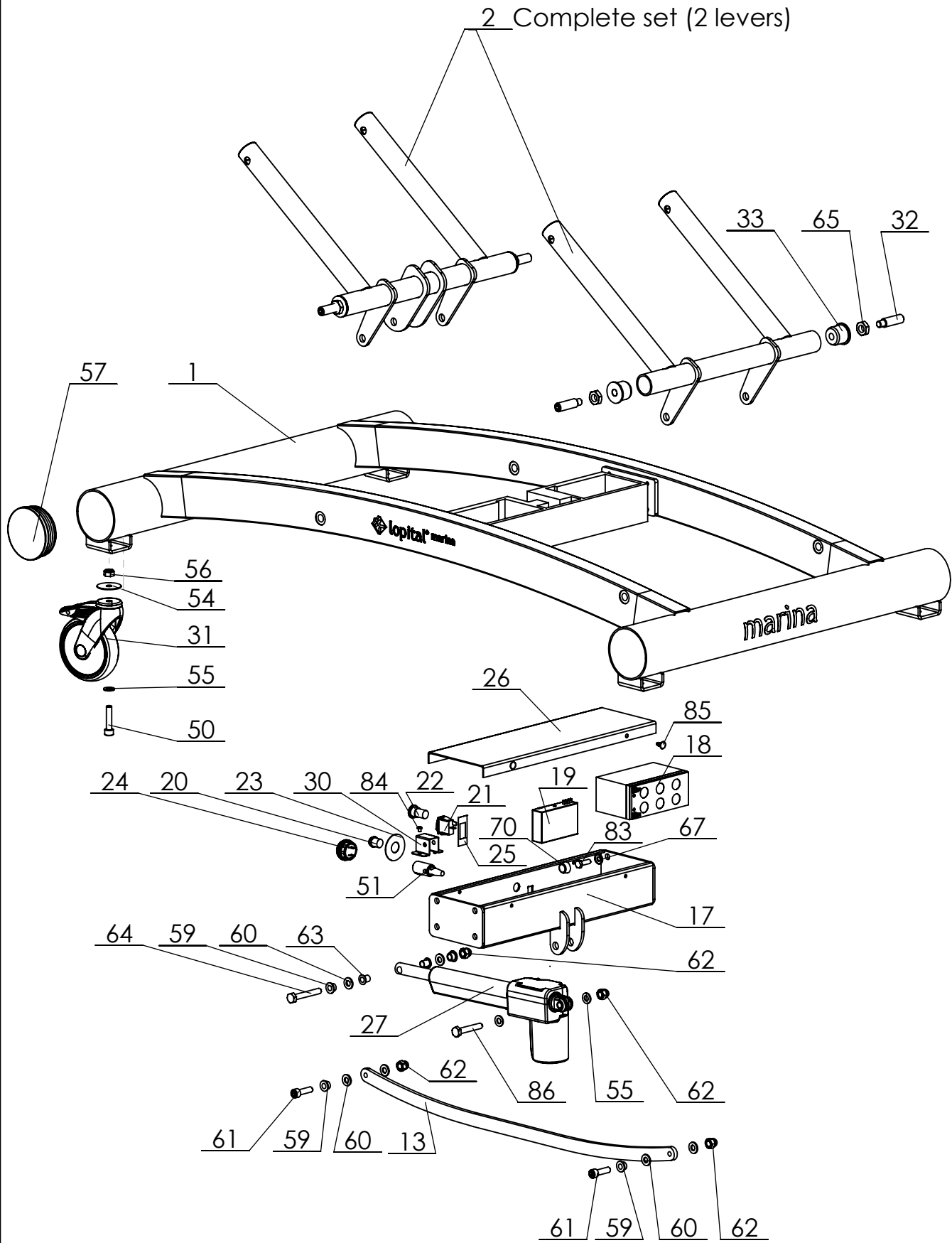

C:\Backup 16-03-2020\SW-Tek\Marina 2300\2300 82x204\

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina Base frame

Copyright reserved by law	Unit of measurement: mm	Lopital Nederland bv	Type nr: 61002350
---------------------------	-------------------------	----------------------	-------------------

## Marina 61002350

Part nr.	QTY.	Art.nr.	Part
1	1	70230001	Chassis
2	1	70230003	Lever set
3	1	70230006	Subframe
4	1	see table last page	Frame
5	2	70230013	Axis holder
6	1	70230014	Tilt lever right
7	1	70230015	Tilt lever left
8	1	70230016	Axis long
9	1	70230017	Axis short
10	1	70230018	Bush
11	2	see table last page	Side rail
12	8	70220023	Clamping plates
13	2	70230025	Pull stroke
14	1	see table last page	Lying plate
15	2	70230012	U-bracket
16	1	70210040	Gas spring
17	1	70235002	Transom electric.
18	1	70560004	Battery Set
19	1	70550013	Controlbox
20	1	70550014	Emergency lowering button
21	1	70550045	Switch On/Off
22	1	70235021	Lock button Yellow
23	1	70550043	Sticker ELD
24	1	70550018	Chassispart charger
25	1	70550044	Sticker on/off
26	1	70235003	Lid transom
27	1	70235004	Actuator TA7
28	2	70210041	Leaf spring incl. cover
29	2	70210039	Lock siderail
30	1	70550021	Bracket coupling
31	4	see table last page	Castor Ø 125 mm braked
32	4	70307001	Custom screw
33	4	70230008	Lever bearing
34	4	70220040	Plug for buffer role
35	4	70220041	Buffer roll
36	24	70220027	Threaded bush
37	2	70210053	Cover gaspring handle
38	1	see table last page	Mattress
39	1	70210048	Pillow

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 61002350

## Marina 61002350

Part nr.	QTY.	Art.nr.	Part
40	1	70210056	Clamping ring
41	1	70210043	Clamping disk
42	1	70210054	Hose (complete)
43	1	70210055	Spout
44	1	70220028	Hose clip
45	1	70210058	Flat seal washer
46	1	70235007	Cable set complete
47	1	70235006	Foam
48	1	70550012	Charger incl. MAG
49	1	70550007	Manual control
50	4	70519935	Cylinder head screw M10 x 45
51	1		Contra plug handcontrol
52	1		Reservoir coupling
53	4		Flanged Slide bearing 18x22x25
54	4		Ring 45x10.5x1.5
55	6		Washer M10
56	4		Locknut M10
57	4		Tube cap Ø100
58	1		Locknut M5
59	8		Flanged slide bearing 10x14x10
60	10		Washer PA M10
61	4		Cylinder head screw M10x35
62	6		Capnut M10
63	2		Flanged Slide bearing 10x12x15
64	1		Hex bolt M10x70
65	4		Low Hex nut M16
66	16		Slide bearing 8x14x10
67	16		Cap ring M8
68	4		Hex bolt M8 x 100
69	8		Locknut M8
70	17		Screw cap M8
71	8		Tube cap Ø40
72	2		Cylinder head screw M5x25
73	4		Washer M5
74	8		Washer M6
75	4		Hexagonal head bolt M6 x 45
76	4		Locknut M6

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina

Copyright reserved  
by law


Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 61002350

## Marina 61002350

Part nr.	QTY.	Art.nr.	Part
77	2		Buffer 20x18
78	6		Washer M8
79	6		Tube cap 50x25
80	4		Tube cap Ø30
81	4		Hex bolt M8 x 55
82	48		Hexagon socket button head screw M5x8
83	8		Hex bolt M8x25
84	2		Hexagon socket button head screw M5x6
85	4		Tree clip
86	1		Hex bolt M10x60

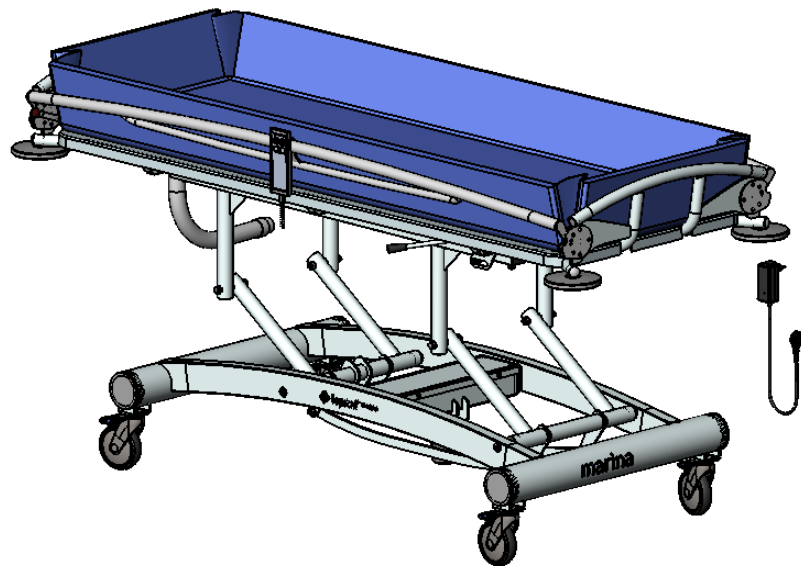

C:\Backup 16-05-2020\SW-Tek\Tek\Marina 2300\2300 82x204\

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina

Copyright reserved  
by law

Unit of measure:  
mm

Lopital Nederland bv

Type nr: 61002350

The parts below differ per Marina variant.

	<b>Article number part</b>	<b>Marina variant</b>
<b>Siderail</b>	80220004 9970221604 9970232002	2300 2305 2310 2350 2355 2360 2330 2345 2380 2385 2320 2370
<b>Lying plate</b>	80220014 80230503 9975233003 9975236001 9970232003	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Mattress</b>	75220016 75230504 75221601 75234503 75232006	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Frame</b>	9970230007 9970230502 9970221602 9970234502 9970232001	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Castor</b>	70519928 70201002 (set p.4)	2300 2305 2320 2330 2345 2350 2355 2370 2380 2385 2310 2360