


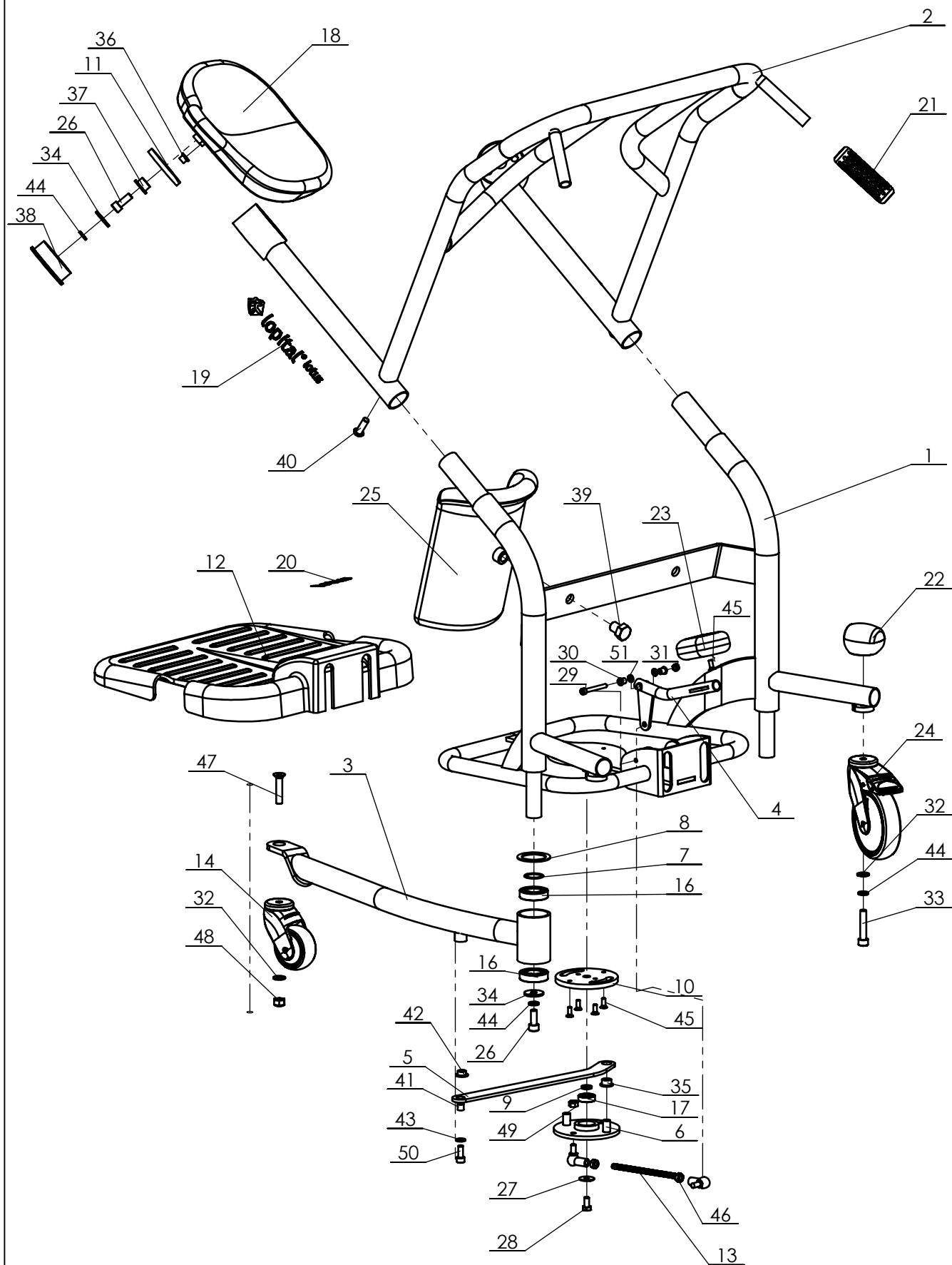

Signed	KA	Date	21-12-2017
Revised	05 /KA	Date	19-01-2018
Revised	06 / KA	Date	23-03-2018

Overview parts Lotus

Lotus 59008100

Part nr.	Default/QTY.	Art.nr.	Part
1	1	70810001	Chassis/Base frame
2	1	70810002	Seat frame
3	1	70810003	Leg set
4	2	70810004	Pedal
5	2	70810005	Rod
6	1	70810006	Turning part
7	2	70810008	Ring s=1
8	2	70810009	Cover hinge
9	1	70810010	Ring s=3
10	1	70810012	Spring plate
11	2	70810013	Slide plate
12	1	70810014	Footplate
13	2	70810015	threaded rod M8 x 140
14	2	70810016	Castor 75 mm unbraked
15	4	70810017	Ball joint M8
16	4	70810018	Ball Bearing RVS 6005-2RS-F
17	1	70810019	Ball Bearing RVS 6001-2RS-F
18	2	70810021	Seat
19	2	70810022	Sticker lopital + lotus
20	1	70810023	Sticker lotus
21	2	70560049	Handgrip
22	2	70560066	Protective cover wheel
23	2	70519920	Pedal cap
24	2	70519928	Castor Ø 125 mm braked
25	2	70802602	Lower leg support
26	4		Cylinder head screw M10x25
27	1		Washer M8 x 25
28	1		Hex bolt
29	2		Cylinder head screw M6x50
30	4		Slide bearing 6x9x11
31	2		Locknut M6
32	4		Washer M10
33	2		Cylinder head screw M10x50
34	4		Ring M10 x 32 (3D)
35	2		Slide bearing 12x16x10
36	2		Slide bearing 8x11x10
37	2		Slide bearing 16x20x13.5
38	2		Insert Ø80
39	2		Hexagon head screw M16x20
40	2		M10x30 Hexagon socket button head screw
41	2		Bush 8,2x10x10

Signed	KA	Date	21-12-2017
Revised	05 /KA	Date	19-01-2018
Revised	06 /KA	Date	23-03-2018

Copyright reserved
by law

Unit of measure-
ment: mm


Overview parts Lotus

Lopital Nederland bv

Type nr: 8100-EV.6

Lotus 59008100

Part nr.	Default/QTY.	Art.nr.	Part
42	2		Slide bearing 10x14x8
43	2		Washer M8
44	6		Nord-Lock M10
45	6		Countersunk head screw M6x14
46	5		Locknut M8
47	2		Countersunk head screw M10x50
48	2		Lock nut M10
49	4		Lock nut M8
50	2		Cylinder head screw M8x20
51	4		Washer M6

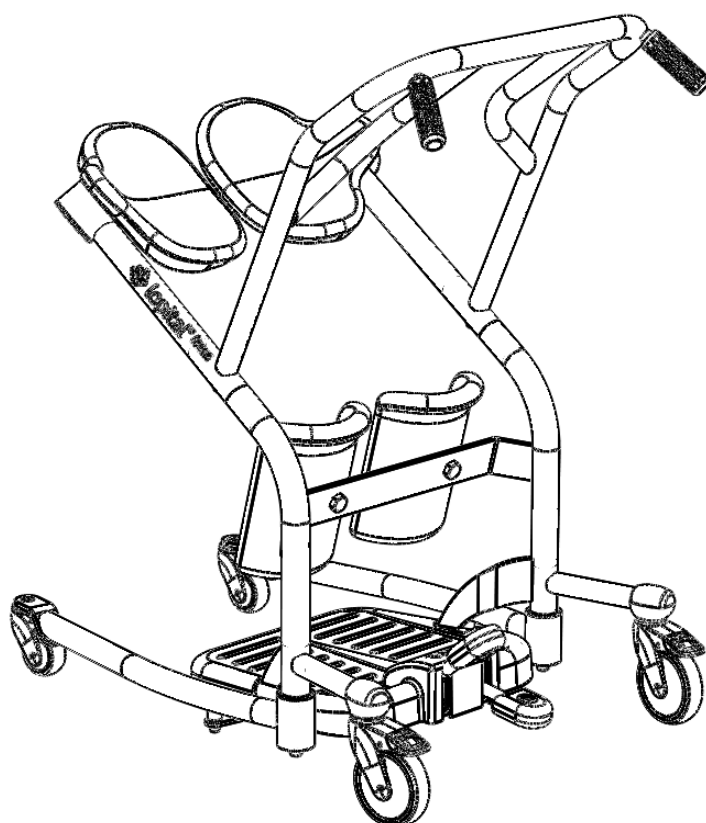

Signed	KA	Date	21-12-2017
Revised	05 /KA	Date	19-01-2018
Revised	06 /KA	Date	23-03-2018

Copyright reserved
by law

Unit of measure-
ment: mm

Overview parts Lotus

Lopital Nederland bv

Type nr: 8100-EV.6